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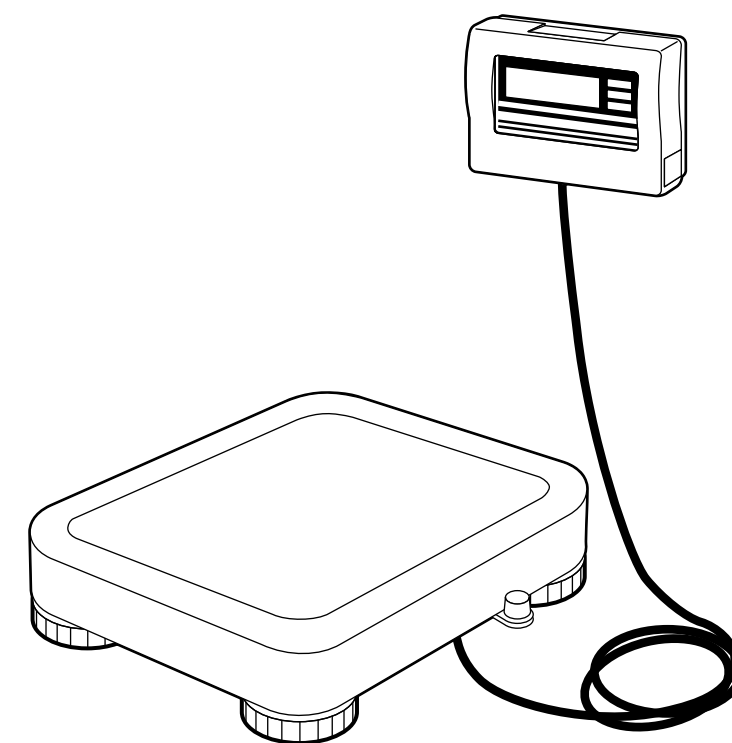
# TANITA

## ELECTRONIC SCALE BWB-800MA SEPARATE INSTRUCTION MANUAL

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Please read this Instruction Manual carefully and keep it handy for future reference.

# Table of Contents

<input type="checkbox"/> Table of Contents.....	3
<input type="checkbox"/> Applications.....	4
<input type="checkbox"/> <b>Safety Notes</b> .....	<b>4</b>
<b>Caution Symbols</b> .....	<b>4</b>
⚠ <b>Warning</b> .....	<b>4</b>
⚠ <b>Caution</b> .....	<b>4</b>
⚠ <b>[Maintenance]</b> .....	<b>5</b>
⚠ <b>[Instructions for Measurement]</b> .....	<b>5</b>
<b>&lt;Usage Conditions&gt;</b> .....	<b>5</b>
<b>&lt;Storage Conditions&gt;</b> .....	<b>5</b>
<b>&lt;Power Source&gt;</b> .....	<b>5</b>
<b>1. Product Assembly and Components</b> .....	<b>6</b>
■ <b>Product Components</b> .....	<b>6</b>
■ <b>Control Panel</b> .....	<b>7</b>
■ <b>Symbols and their Meanings</b> .....	<b>7</b>
<b>2. Preparation for Use</b> .....	<b>8</b>
■ <b>Level Adjustment</b> .....	<b>8</b>
■ <b>Installation of the Equipment</b> .....	<b>8</b>
■ <b>Prepare Power Source</b> .....	<b>9</b>
<b>3. Operating Instructions</b> .....	<b>10</b>
■ <b>Measurement Procedures</b> .....	<b>10</b>
■ <b>Automatic Power Off</b> .....	<b>10</b>
<b>4. Troubleshooting</b> .....	<b>11</b>
■ <b>Possible problems</b> .....	<b>11</b>
<input type="checkbox"/> Specifications .....	Back cover

# Applications

- This equipment has been calibrated as a precision weighing instrument and can be used to certify weights and/or business transactions.
- It can be used to obtain reference data during medical examinations, such as periodic checkups, and can help the prevention of obesity.

# Safety Notes

## Caution Symbols

In this Instruction Manual and the descriptions on the equipment, there are various caution symbols that will enable you to use this equipment safely and correctly so that harm to you and other people and loss of your property will be prevented.

The symbols and their meanings are as follows. Please understand them well before proceeding with the Instruction Manual.

	<b>Warning</b>	This symbol indicates the risk of death or serious injury if the safety instructions are ignored and the equipment is misused.
	<b>Caution</b>	This symbol indicates the risk of physical injury and/or damage to the equipment if the safety instructions are ignored and the equipment is misused.
		This symbol indicates general precautions that should be taken when using this equipment.

## Warning

- To prevent electric shocks,  
Do not insert or remove the power plug with wet hands.
- To prevent fire hazards,  
Always use a correctly wired 230V AC not multiple extension cords.

## Caution

- To prevent the spread of infections,  
Always clean the Weighing platform after use. Sterilize the platform with an appropriate alcohol-based cleaning fluid before each use.
- Installation of the Weighing Platform  
Ensure the weighing platform is placed on a level and stable surface. If the equipment is used when the Platform is unstable, for example because not all the feet are on a level surface, there is a risk that it will topple over and it will be impossible to obtain an accurate measurement.  
To avoid the risk of injury and damage to the equipment, never jump on the platform.
- How to handle the equipment when moving it  
Since the Platform is heavy, please lift it by holding the Platform firmly with both hands so that you do not drop it.
- AC Adapter  
Ensure that the adapter supplied (model A30930G) is always used with this equipment. Other adapters may cause malfunctions.
- Battery  
When changing batteries, ensure that the new batteries are inserted with the correct polarity ⊕/⊖. If the polarity is incorrect, then the batteries may leak and damage the equipment.

- The circuit board incorporates a fuse (100mA, 125V) for electronic equipment. If devices other than the original AC adapter are used or a large electrical current is applied, the fuse will be blown for safety purposes. If the fuse is blown, the unit must be inspected and repaired. As the fuse forms a fixed part of the unit and cannot be easily disassembled by the customer, please consult your nearest Tanita sales office or agent when the fuse needs to be replaced.

## [Maintenance]

**This is a precision manufactured and accurately calibrated product. Please observe the following instructions.**

- Never disassemble or adjust the equipment, as this may cause malfunctions. The equipment should only be inspected in accordance with the regulations in each country.
- When not in use for a long time, unplug the AC Adapter from the wall socket.
- When not in use for a long time, remove the batteries before storing the equipment.
- Do not wipe the equipment with corrosive chemicals (benzine, cleaner, etc.). Please use a neutral detergent to clean the equipment.
- Avoid subjecting the equipment to excessive shocks or vibrations.
- When disposing of this unit, please do so in accordance with the prevailing regulations in each country.
- If an unauthorised person attempts to disassemble or repair any of the part, the warranty will become invalid. When the unit malfunctions, please consult your nearest Tanita sales office or agent.

## [Instructions for Measurement]

**Please observe the following instructions for accurate measurement.**

- Measurement may not be possible on a surface that is strongly vibrating. In such cases transfer the equipment to a stable surface with little or no vibration.
- When the equipment has been transferred to another location with a temperature difference of more than 20°C, leave at least two hours before using.
- While taking measurements, please keep any persons using transmitters, such as a mobile phone, away from the unit to avoid causing margin errors.

### <Usage Conditions>

Temperature Range for Use	: 0°C / 35°C
Relative Humidity	: 30% / 80% (without condensation)

### <Storage Conditions>

Temperature Range of Environment	: -10°C / 50°C
Range of Relative Humidity	: 10% / 90% (without condensation)
<b>To avoid malfunctions, do not store the equipment in direct sunlight, significant temperature changes, the risk of dampness, a large amount of dust, in the vicinity of fires or where there is a risk of receiving vibrations or shocks.</b>	

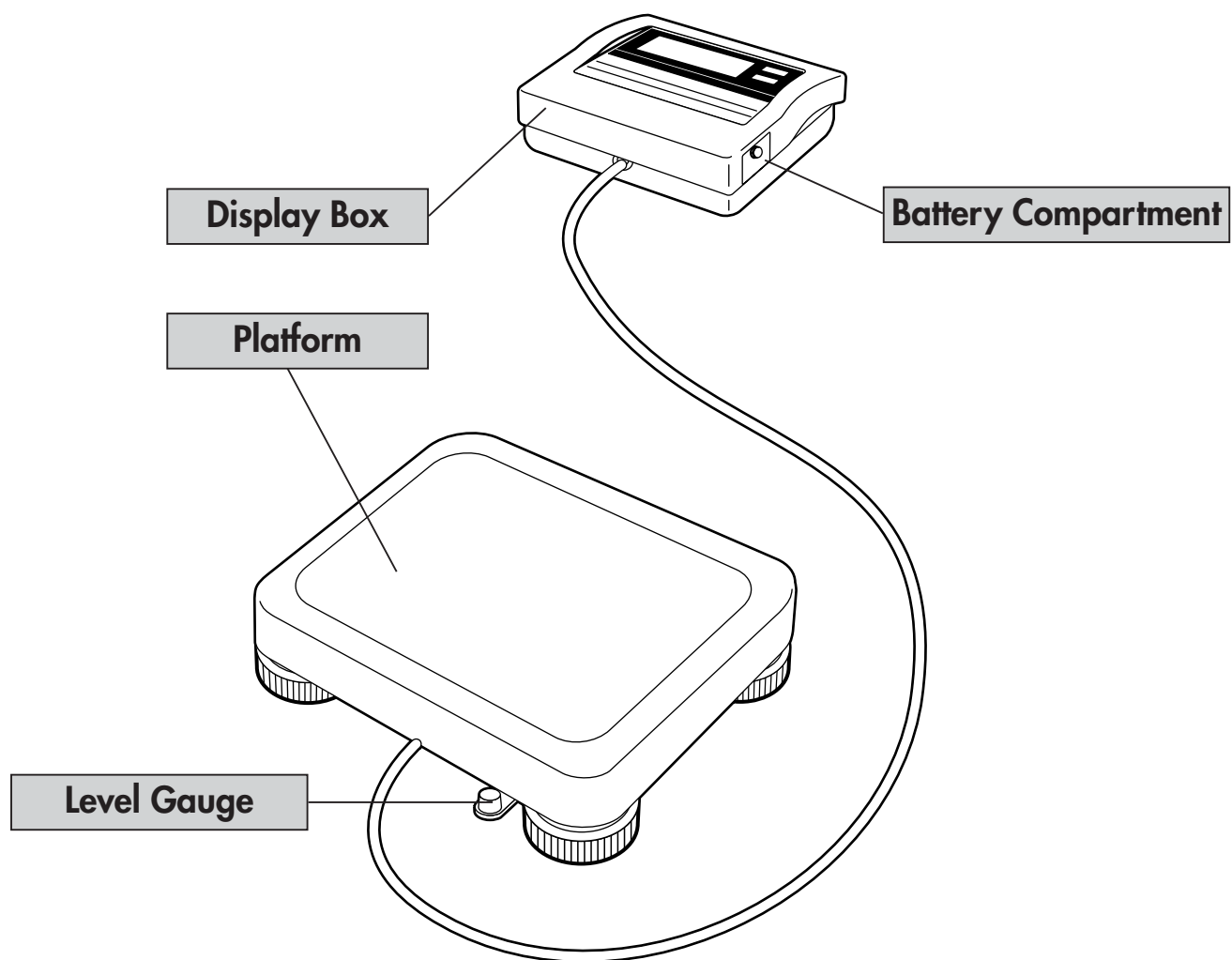
### <Power Source>

	Value Range
Voltage Range	230VAC
Frequency Range	50Hz
Electric Current Range	65mA

# 1. Product Assembly and Components


## Product Components

Please verify that the following components are in the package.


## Accessories

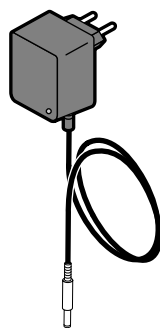AC Adapter  
MODEL:A30930G


Bracket for Wall Installation

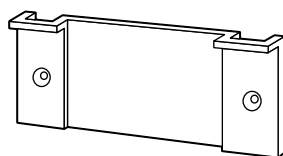Fixing Screw for the Bracket (x 2)

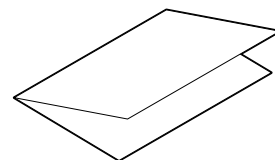Instruction Manual

## Control Panel

① Turn on the power for the Display Box and reset the display to zero.

② Turn off the power for the Display Box.

③ **STABILIZED** When the load is stable, and arrow will be displayed to the left of "STABILIZED".  
\* Due to the precision of this unit, the arrow may not appear immediately. This does not mean that the product is defective.

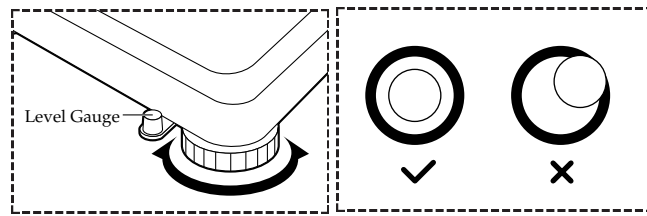
④ When there is no load on the platform, and the unit has zeroed, then an arrow will appear next to the words "→0←".  
\* When a load is placed on the Platform, then the arrow will disappear.

## Symbols and their Meanings

	: Display Unit On.		: Display Unit Off.		: Direct current
	: Class II Equipment		: Type B applied part		: Zero reset
	: Battery change is required		: Caution. Refer to the attached notes.		: Conformity with Medical Device Directive 93/42EEC

# 2.Preparation for Use

## Level Adjustment


\*Adjust the height by rotating the adjustable feet.

For accurate measurement, place the unit on as flat a surface as possible and adjust the level with the adjustable feet so the bubble in the level gauge is in the center of the frame.

### Caution

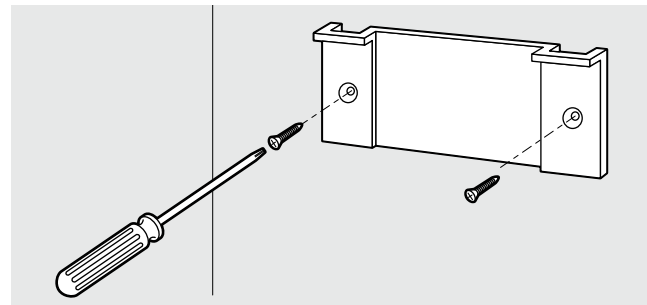Place the equipment on a level surface. If the equipment is not stable, for example because not all the feet are on a level surface, there is a risk that it will topple over and it will be impossible to obtain an accurate measurement.


## Installation of the Equipment


The Display Box can be placed on the table or hung on the wall.

<When hanging on the wall>

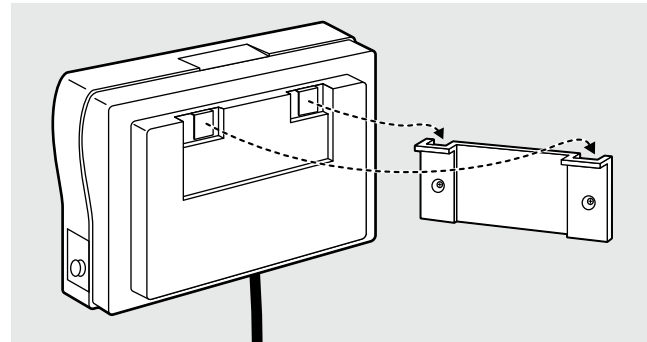① Attach the Bracket for Wall Installation onto the wall using the screws through the two holes in the Bracket. Ensure that the protruding edges of the Bracket are uppermost.

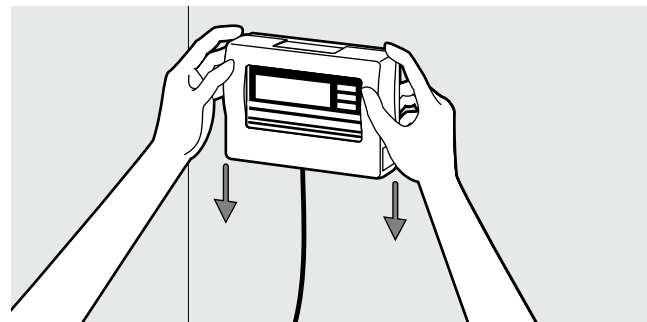
② Mount the Display Box onto the Bracket, inserting the hooks of the Display Box into the protruding edges of the Bracket.

③ Ensure that the Display Box is firmly fixed to the Bracket by gently pulling down on the Display Box.

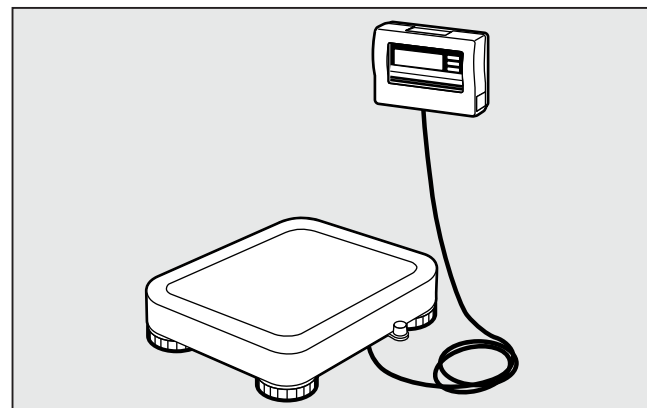
- If the Display Box is not fixed sufficiently, there is a danger that it may fall down.
- Avoid installing the Display Box in a place where users may step on or get caught by the cable.
- Do not swing the Display Box by the cable.

## Prepare Power Source
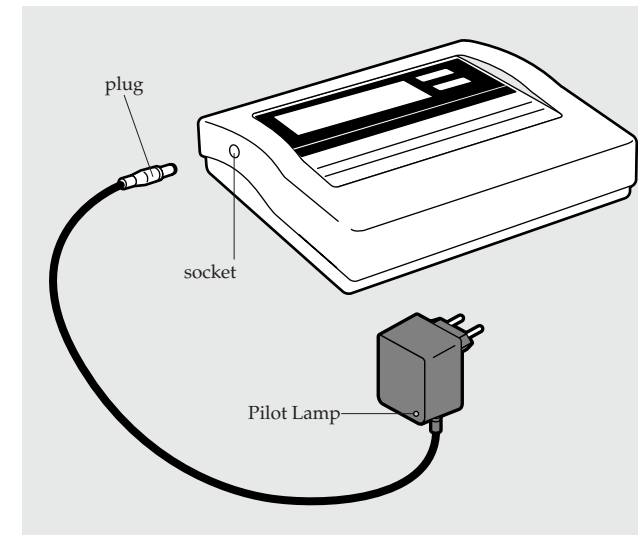
### ① When using the AC Adapter

Insert the cable plug for the AC Adapter into the socket on the side of the Display Box, and insert the AC Adapter plug into a 230V AC mains socket.

Then confirm the Pilot Lamp on the AC Adapter lights up.


### Warning

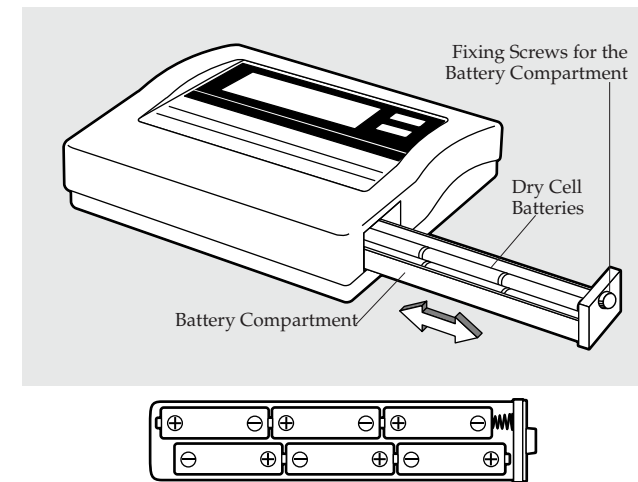• To avoid electric shocks, do not insert and remove the plug with wet hands.

### Caution

• Use only the AC adapter (MODEL: A30930G) specified. Using an AC Adapter other than that specified may cause malfunctions.

### ② When using dry cell batteries

Unscrew the Fixing Screws for the Battery Compartment on the right side of the Display Box and pull out the Battery Compartment. Place dry cell batteries in the correct positions as instructed inside the Battery Compartment and fix the Battery Compartment with the Fixing Screws.

If **Lo** is displayed on the Display and **s** appears next to the **B+** symbol, please replace all the six batteries with new ones.


- Please change the batteries carefully to avoid dropping them on your feet.
- Ensure that the batteries are inserted with the correct polarity  $\oplus / \ominus$ . If the polarity is incorrect, then the batteries may leak and damage the equipment.
- When not in use for a long time, remove the batteries before storing the equipment.
- When the equipment has been transferred to another location with a temperature difference of more than 20°C, leave at least two hours before using.

# 3. Operating Instructions
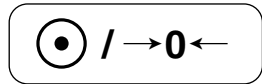
## ■ Measurement Procedures

- ⚠ When the equipment has been transferred to another location with a temperature difference of more than 20°C, leave at least two hours before using.
- While taking measurements, please keep any persons using transmitters, such as a mobile phone, away from the unit to avoid causing margin errors.

1. Press the  Key

 will flash on the Display as shown on the left, then  will be displayed and ◀ will appear next to "→0←".

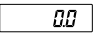
2. Step on the Platform and Start Measurement




Please step on the Platform.

An arrow (◀) will appear next to "STABILIZED" and the weight will be displayed.

3. Measurement is Completed

Please step off the Platform. Press the  key and the power will be turned off.

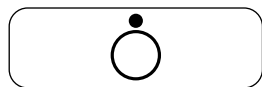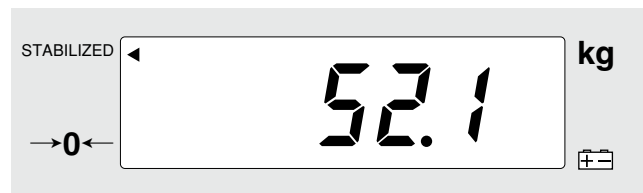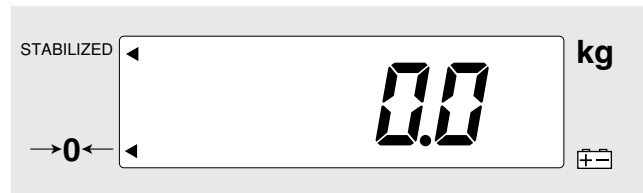\* If you continue measurement, please step off the Platform and after confirming  is displayed, then start measurement again.

## ■ Automatic Power Off

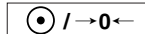
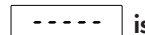
This equipment has an automatic power off function that automatically turns off the power in order to reduce battery consumption.

If the display does not change for 20 minutes, the power will be automatically turned off.

# 4. Troubleshooting

## ■ Possible Problems

If problems occur, please refer to the following instructions before you ask for repairs.

Problem	Solution
The unit does not turn on when the  key is pressed.	<ul style="list-style-type: none"> <li>• When Using with the AC Adapter Ensure that the AC Adapter cable is properly connected with the Display Unit. Ensure that the AC Adapter is correctly inserted in the mains socket.</li> <li>• Using dry cell batteries The batteries may have run out. Confirm that the equipment is working by using the AC Adapter or new batteries.</li> </ul>
Measurement is not correct.	<ul style="list-style-type: none"> <li>• Please confirm that all the adjustable feet are resting on a level surface.</li> </ul>
Measurement does not become stable. Display turns on and off.	<ul style="list-style-type: none"> <li>• Measurement may be impossible in the case of excessive vibration. Use the equipment in another place.</li> </ul>
 is displayed.	<ul style="list-style-type: none"> <li>• It is possible that the maximum weight capacity has been exceeded. Please stop measuring.</li> </ul>
 is displayed.	<ul style="list-style-type: none"> <li>• Do not stand on or put objects on the Platform before starting measurement. Start measurement after confirming there is nothing on the Platform.</li> </ul>

# Specifications

<b>Model</b>	BWB-800MA	
<b>Power source</b>	AC adapter Centre minus MODEL A30930G CLASS 2 Input Voltage : 230VAC 50Hz 65mA Output Voltage : 9VDC±0.45VDC Rated Current : 300mA No Load Input Voltage : 12VDC±0.6VDC Alkali battery (LR6) x 6, (Not included)	
<b>Power Consumption</b>	0.25W max.	
<b>Measurement System</b>	Electrical resistance method	
<b>Maximum Capacity</b>	200kg	
<b>Minimum Graduation</b>	0.1kg	
<b>Accuracy at first calibration</b>	±0.1kg	
<b>Weighing Range</b>	2kg - 200kg	
<b>Display</b>	LCD (Liquid crystal) (Height of numerals 25mm)	
<b>Temperature Range of Usage</b>	0 to 35°C	
<b>Relative Humidity</b>	30 - 80% (without condensation)	
<b>Size</b>	<b>Platform</b>	301 x 336 x 88 (mm)
	<b>Display box</b>	160 x 200 x 60 (mm)
<b>Battery life</b>	approximately 100 hours of continuous use.	

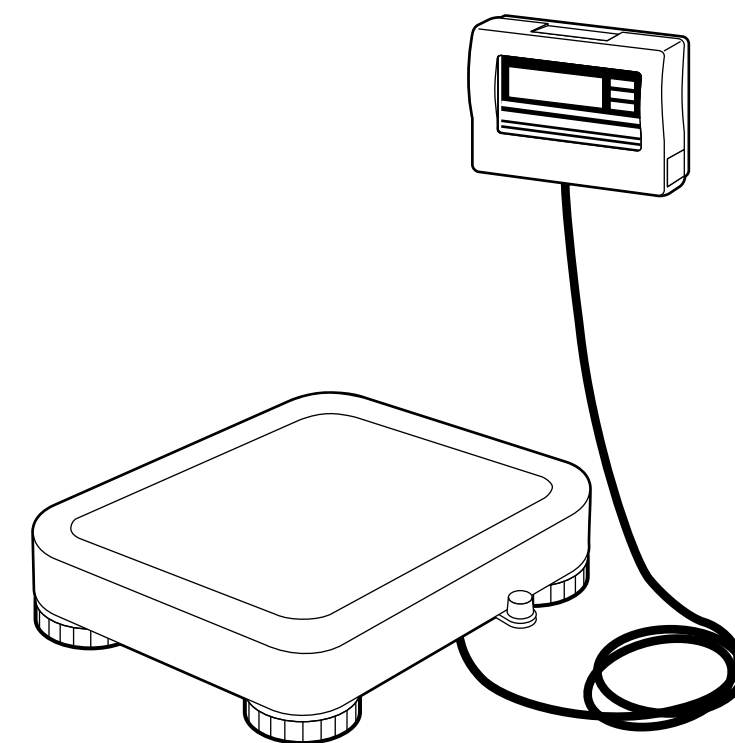
# TANITA

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## BWB-800MA

## SEPARATE

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The product meets all the requirements under the three standards below:

Safety Standard : EN60601-1:1990, AM No.1:1993, AM No.2:1995  
IEC60601-1:1988, AM No.1:1991, AM No.2:1995  
EMC Standard : EN-60601-1-2:1993  
IEC60601-1-2:1993  
Medical Device Directory Standard : 93/42/EEC:1993

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